


Installation Guide

QuickFit™


Earmold Tubing

1. QUICKFIT EARMOLD OVERVIEW

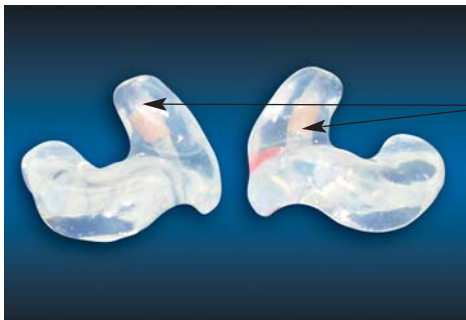


Coupler
(tube retainer)

Tube Outlet

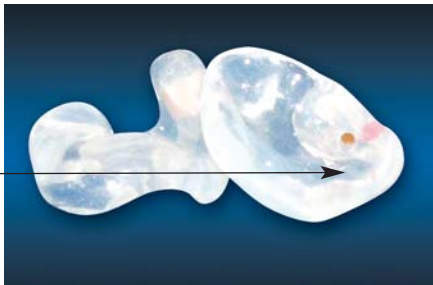


2. PREPARATION OF NEW TUBE

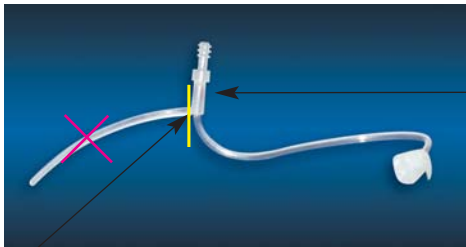

Red marking for right/Blue for left

Cut off original retainer string at this point

TUBE INSERTION

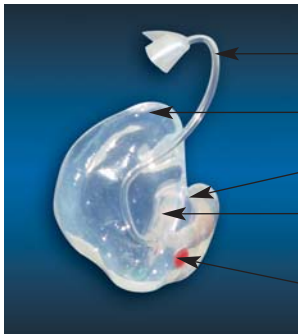

Matching color tube

Helix (top) side

Canal of Earmold

Tube barrel flush to earmold surface

Side ID dot – Red for right, blue for left

Note: Make sure that the position of the sound tube is as shown.

4. MOUNTING QUICKFIT DEVICE TO EARMOLD AND ON THE EAR:



5. USING QUICKFIT EARMOLD AS AN EAR PLUG



Remove sound tube and insert cone-shaped plug into the hole.

6. USING QUICKFIT EARMOLD AS AN OPEN EAR DEVICE



Dome tip or
Foam tip



Retention string in
concha bowl

Tip in ear canal



**E.A.R./Insta-Mold, Inc. ■ P.O. Box 18888
Boulder, CO. 80308 U.S.A. ■ (800) 525-2690
(303) 447-2619 ■ E-mail: info@earinc.com
www.earinc.com ■ www.freehearingtest.com**